



DAILY CHECK OF FM/DM SYSTEM

1. After performing a check of the hearing aid, attach the FM/DM receiver to the hearing aid.
2. Turn on the FM transmitter and check that it is on the correct channel. (If the child has DM technology, there are no channels - just turn the transmitter on.)
3. Put the stethoscope on the hearing aid mold and turn on the hearing aid and listen for beeps indicating that the receiver has synced.
4. Speak into the microphone. Listen that the signal is clear without white noise, crackling, intermittency, etc.
5. Put hearing aids back on student's ear and have them repeat the Ling sounds (or ask them questions) with the transmitter on.
6. If any issues, please call your consultant for help!

It is important for an adult with normal hearing to listen to a student's amplification to ensure that it is functioning adequately. The student may check the battery but should not do a listening check as he/she should not be responsible for detecting problems with the equipment. In addition, simply asking the student if he/she can hear while he/she is wearing hearing aids is not sufficient to check the functioning of the equipment.